


Operation


Assures that there are no people present in the Controlled Access Location.

- Only trained people is permitted to use this product.**
The product needs to be used on flat ground, the wheels need to be locked. When moving the lamp, the arm needs to be turned back to the housing and locked.
The key and remote controller should be kept with trained people.

1 Rotate the lock, and open the door

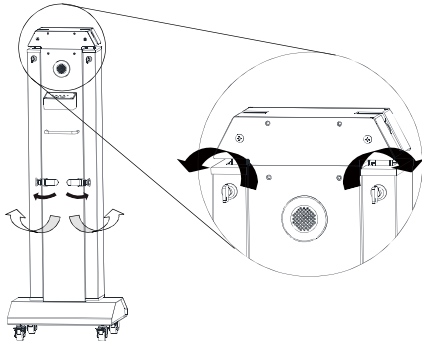

2 Rotate the UV C lamp arm to Appropriate Angel by handle(30°, 60°, 90°, 120°, 150°, 180°)

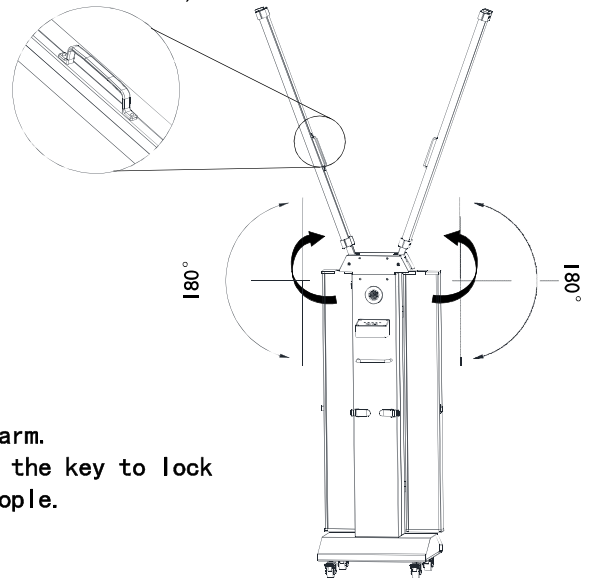

The key should be kept with trained people.

Do not hold the lamp, use red handle to rotate the arm.
 After using, rotate the arm back to housing and use the key to lock the housing. The key should be kept with trained people.

3 Power

Each press operation interval is within 30 sec.

Method One: Control Panel

1. Plug in, and long press Power button for 3 seconds.
2. Press the Timer button to select the disinfection time. Each Press move to next time zone.



3. Press the Power button again, product will delay 30 seconds and power on the UV-C lamp.
4. After the timing is over, the UV-C lamp will auto power off.
5. If need to power off the UV-C lamp during working procedure, press the Power button again.

Method Two: Remote Controller

1. Plug in Product, Press in sequence 15min -1440min power on button on remote controller to unlock the product.
2. Press the Timer button, select the disinfection time.(Optional: 0-1440 minutes)
3. Press the Power button again, product will delay 30 seconds and power on the UV-C lamp.
4. Product will delay 30 seconds and power on the UV-C lamp.
5. After the timing is over, the UV-C lamp will auto power off.
6. If need to power off the UV-C lamp during working procedure, press the Off button.

(Remote control pairing method: press button A minutes and B minutes for 3 seconds at the same time, press "OFF" to turn it OFF and "ON/TIMER+" to turn it ON , the remote controller pairing is complete)


4 After using, rotate the arm back to housing and use the key to lock the housing. The key and remote controller should be kept with trained people.

In case of an accidental lamp breakage, ventilate the room for 30 minutes and remove the parts, preferably with gloves.

Fault Handling

Fault	Handling
The power indicator is on, and the Lamp does not work	1. Rotate the lamp tube so that the lamp tube is in good contact with the lamp holder 2. Replace the lamp. 3. If the lamp tube is still not on, please contact the dealer
The power indicator is not on, and the lamp does not work	Please contact the dealer.

Electrical schematic diagram

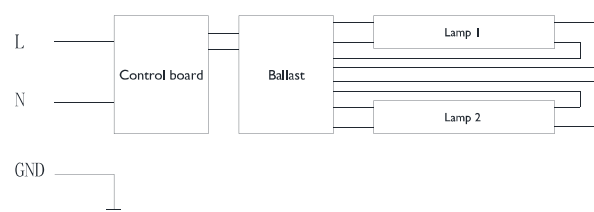


Lamp replacement

Only trained people is permitted to replace the lamp

Please Shut off the power supply and unplug product before changing the lamp! Please ensure to put on gloves when replacing the lamp.


1

Twisted UV lamp

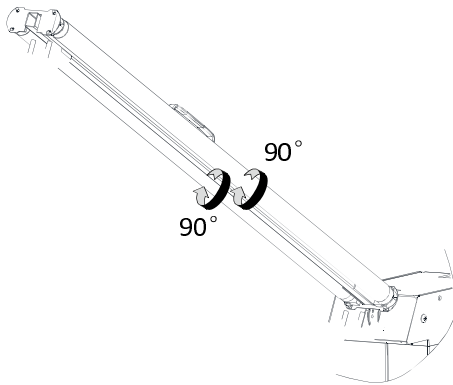

2

Remove the lamp

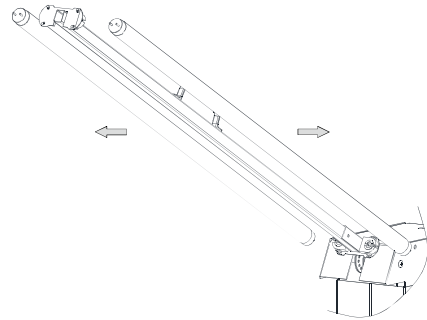

3

Install the lamp

