



Lucent LED Operatory Light Quick Reference Installation Guide

ASSEMBLING THE HEAD TO THE ARM

Remove adaptor
Bushing

Attach bushing
to light head



ELECTRICAL CONNECTIONS


Connect wire
harness

Secure light
head to arm



alandental.com

Light Handles

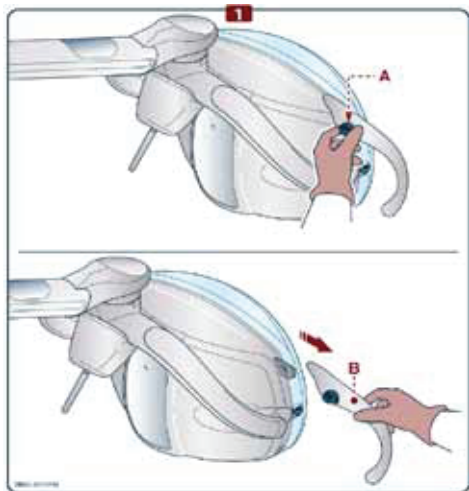

Switch on / off / Adjustment

- To switch the lamp On and Off, press and release the switch to the LEFT or RIGHT side.

When the lamp is switched On / Off, the device will beep (once).

DON'T push and pull the switch. It only moves in a left and right direction.

- Right & Left Light handle
- Light switch toggle extension
- Light shield wrench
- Light bushing pre installed into arm.

Signature _____ Date _____