Teeth Bleaching System

MD-666
User manual

Teeth Bleaching System
MD-666

Catalogue

1. Safety..... 2

2. Assembly Instruction Illustration..... 3

3. Product Features..... 8

4. Product Specification..... 8

5. Content..... 9

6. Operation..... 9

7. Remote control manual..... 10

8. Liability..... 11

Safety

PLEASE NOTE

Prior to installation and start-up of the unit, carefully read the instructions provided herein. As with all technical devices, the proper function and safe operation of this unit depend on the user's compliance with the safety recommendations presented in these operating instructions.

 CAUTION: It is important to observe the following precautions and procedures:

Do not shine the light directly onto or toward unprotected gingival or skin.

 CAUTION: Persons with a history of photosensitivity or who are using photosensitizing

drugs should not be exposed to light from this unit.

 CAUTION: This equipment is not suitable for use in the presence of flammable anesthetic

mixtures with air or nitrous oxide.

 CAUTION: Please do not look into emitted light by the machine when it is in use

or it will hurt your eyes. Please be sure to wear goggles.

 CAUTION: The angle of bend in use please do not exceed 180 degrees, or easily

broken elbow, it may cause noise in the future.

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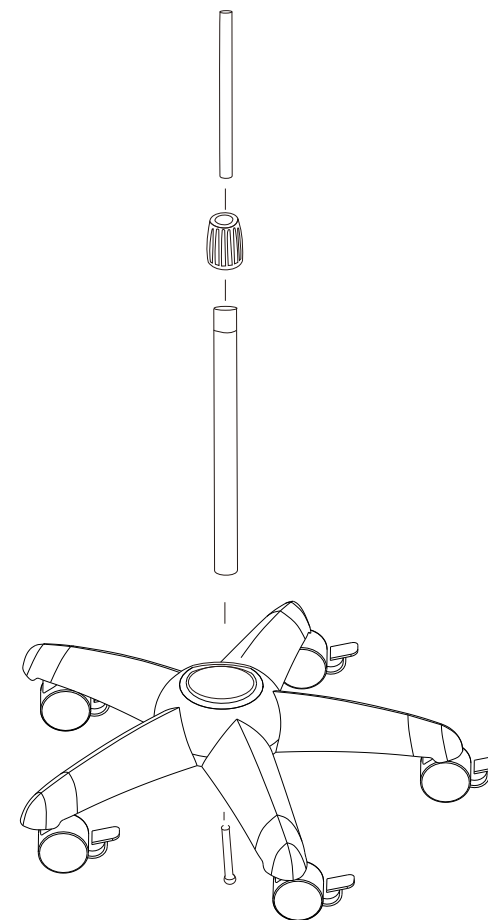
Assembly Instruction Illustration


(1)


Warning:
 When this screw is loose, the spring will be come out

Assembly Instruction Illustration

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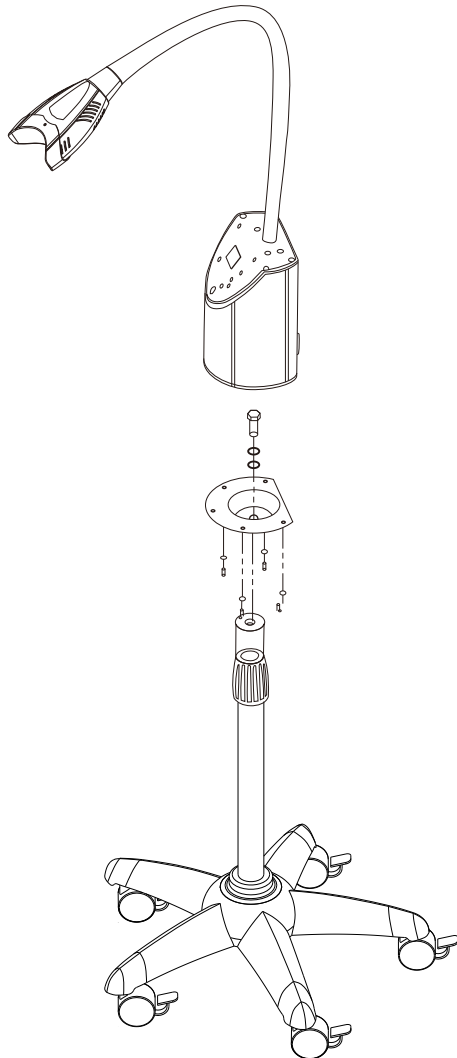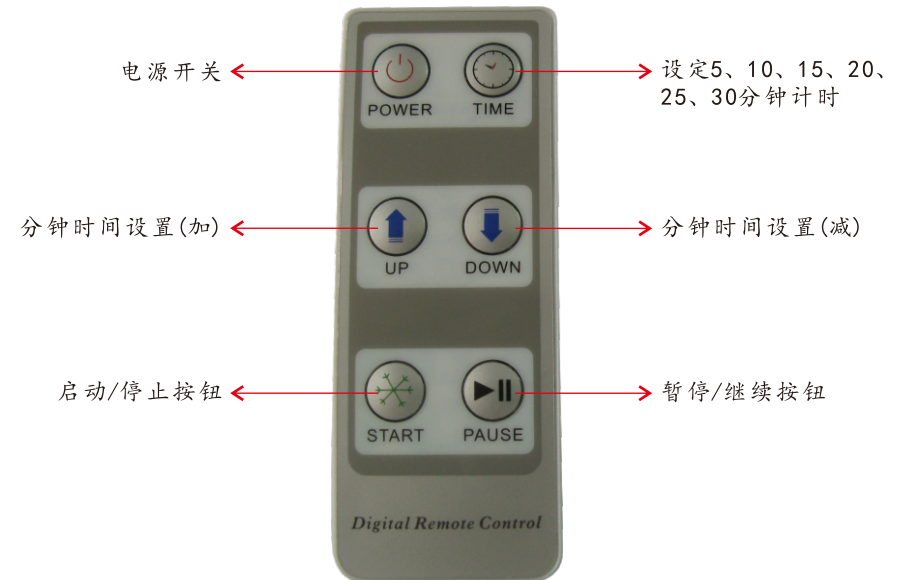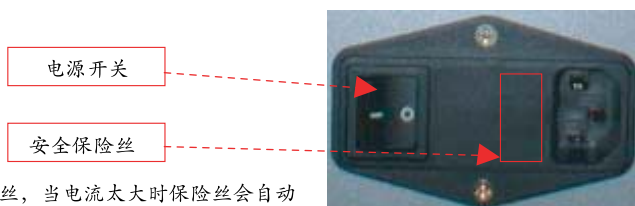
Assembly Instruction Illustration

1. Twist the wheels to the feet, and twist tightly with a wrench

(1)

2 . Fix 5 feet onto the settled seat.

(2)

Assembly Instruction Illustration

3.Fixed tray aim at settled seat screw holes, and fix screws.

(3)

4.Put pole 1 with decorative set placed on the settled seat and support with hand.

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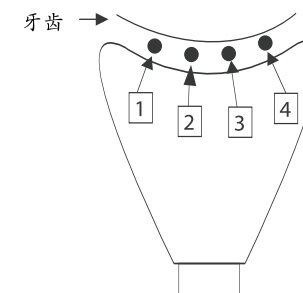
5. Turn over the feet and fix settled
seat and pole with long screw.
(if the screw haven't twist tightly,
the pole will shake)

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(6)

6. Put pole 2 into pole 1 and
twist nut tightly.

(6)


Product Features

- High efficiency goose pipe design, adjust angle at random, convenient for use.
- Combination of high power BLUE led.
- Tunable bleaching time with microprocessor-controlled timer presets.
- Digital indicator with audio feedback.
- Smart auto-select power: 100 to 240 Volts AC.50/60 Hz for global use.
- High Speed Multi-Arch Teeth Whitening System.
- Small base size design, suitable use in dental clinic
- High Uniformity of Light Output
- High sensitive infrared remote control setting function.
- Enclosed disposable sheath, prevent cross- infection, clean and convenient, let patient feel relieved.

Product Specifications

- Broad Spectrum: 420nm~500nm(Host Lamps)
- Light Output Power: Up to 2000 mW/cm²
- 100/240 Volt, 50/60Hz
- Weight: 4.7Kg
- Max. Working Radius: 50cm
- High Uniformity Of Light Output:

Distance between Teeth and Illuminator(mm)	Uniformity (Point 1to Point 5)
10mm	Above95%
20mm	Above90%
30mm	Above85%
40mm	Above80%
50mm	Above75%

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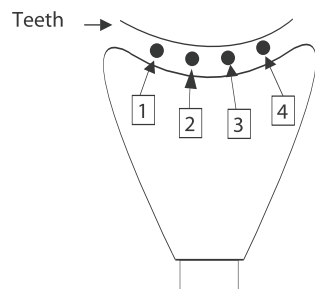(2)


Content

MD-666	x 1 PC
Power Cord	x 1 PC
Glass	x 2 pairs
Sheath	x 20 PCS
Mouth retractor	x 1 PC
Remote control	x 1 PC
User manual	x 1 PC

Operation

Turn on the Teeth Whitening Accelerator:

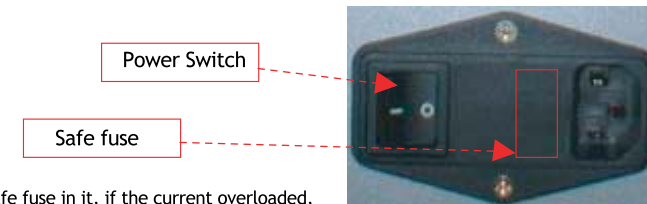- a.) Plug in the Teeth Whitening Accelerator.
- b.) Turn on the power switch


We fixed safe fuse in it, if the current overloaded, it will cut off immediately, when you change new fuse, everything will work smoothly as usual

Operating the Teeth Whitening Accelerator:

- a.) **Start/stop Button:** Turn on/off the light by pressing the "Start/stop" button
- b.) **Pause Button:** Stop the time counting and emitting light by pressing the "Pause" button.
The remaining whitening time will sparkle on the digital indicator .
Press the "Pause" button again, light will emit and time will count continually
- c.) **UP/Down Button:** Use "UP" or "Down" button to set whitening time.
5 minutes counting with memory, the longest whitening time is 30 minutes each cycle

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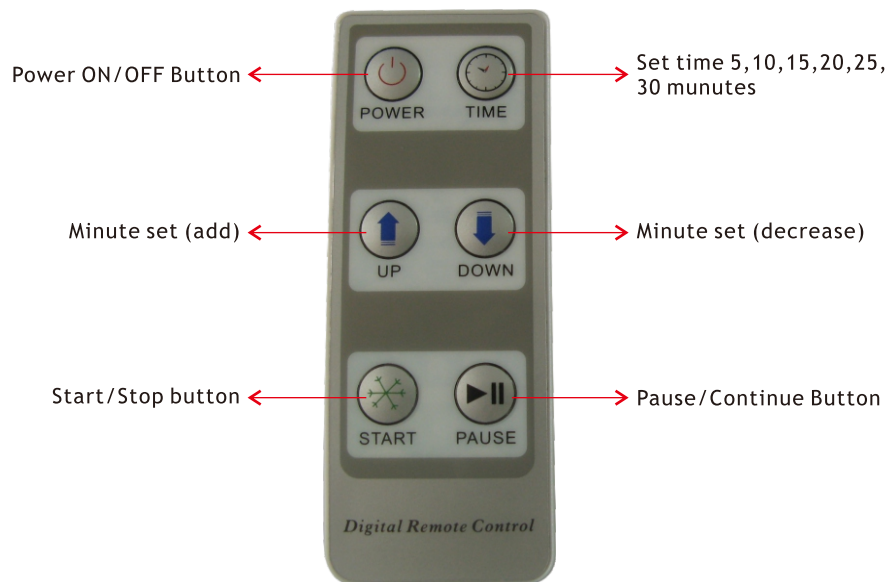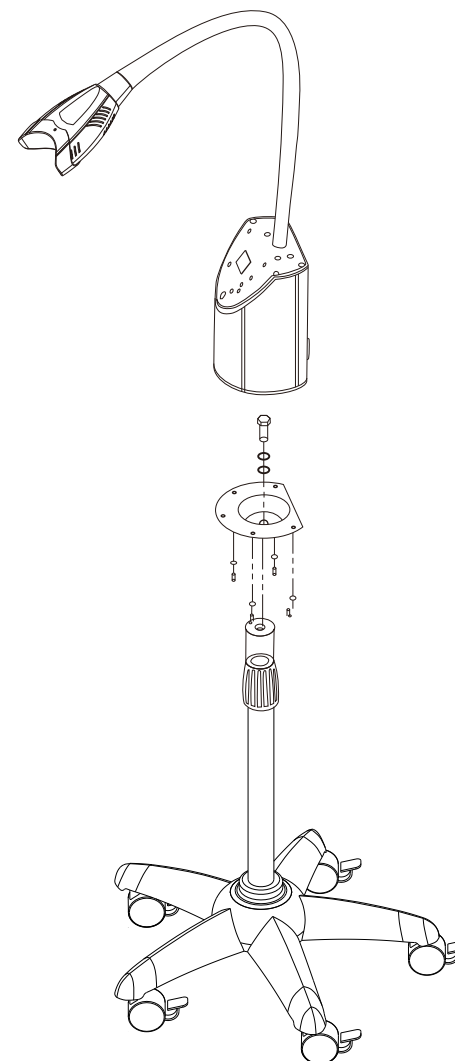
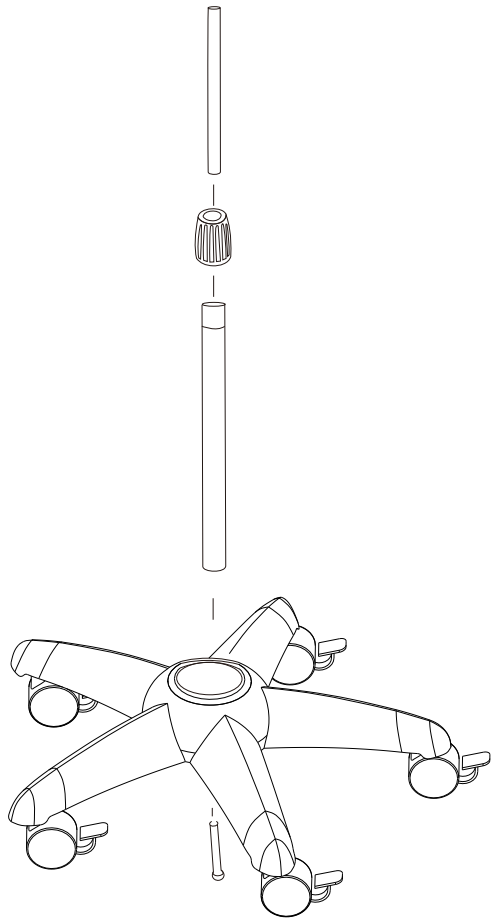
Sound Instruction

- a). After starting, once the button is pressed, a beep sounds.
- b). After starting, when running to the last minute, there will be a beep sound.
- c). After completion, there will be a long beep.

Cooling Setting

- a.) To guarantee all LEDs could work at the normal temperature when whitening process, we fix a high efficiency and quite fan to expel *Heat*.
- b.) Besides, we built-in smart temperature protection, when it meets high temperature will shut down power automatically, after cooling, all can work properly.

Liability

It is responsible for the effects on safety, reliability and performance of this product only if:

- The electrical installation of the relevant room complies with the requirements.
- The equipment is used in accordance with these instructions for use.

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