

PULIHUA PLH-230

Wireless Dental Cordless Hygiene

Prophy polishing Handpiece+2

Prophy Angles

User Manual



Please read this instruction manual carefully before use, and file it for future reference.

For dental use only.

1.Product Overview

- Thanks for using the PLH - 230 Dental Polishing Machine(Motor).
- This portable dental polishing machine is a precision tool specially designed for dental treatment and restoration. It features small size, light weight, flexible operation, adjustable speed and low noise, and is suitable for grinding, polishing and trimming various dental materials.
- This equipment should be operated by qualified or approved personnel. The operator should have specialized application skills and work in accordance with the manual.
- The equipment must not be used in an environment where flammable anesthetic gases are mixed with air or nitrous oxide.

2.Technical Specifications:

Packaging Size: 195*127*66mm

Gross Weight: 432g


Battery: 800mAh

Work Environment Temperature: 5°C-40°C

Relative Humidity: 10%-100%

Atmospheric Pressure: 50~106KPa

3.Main Directions For Use

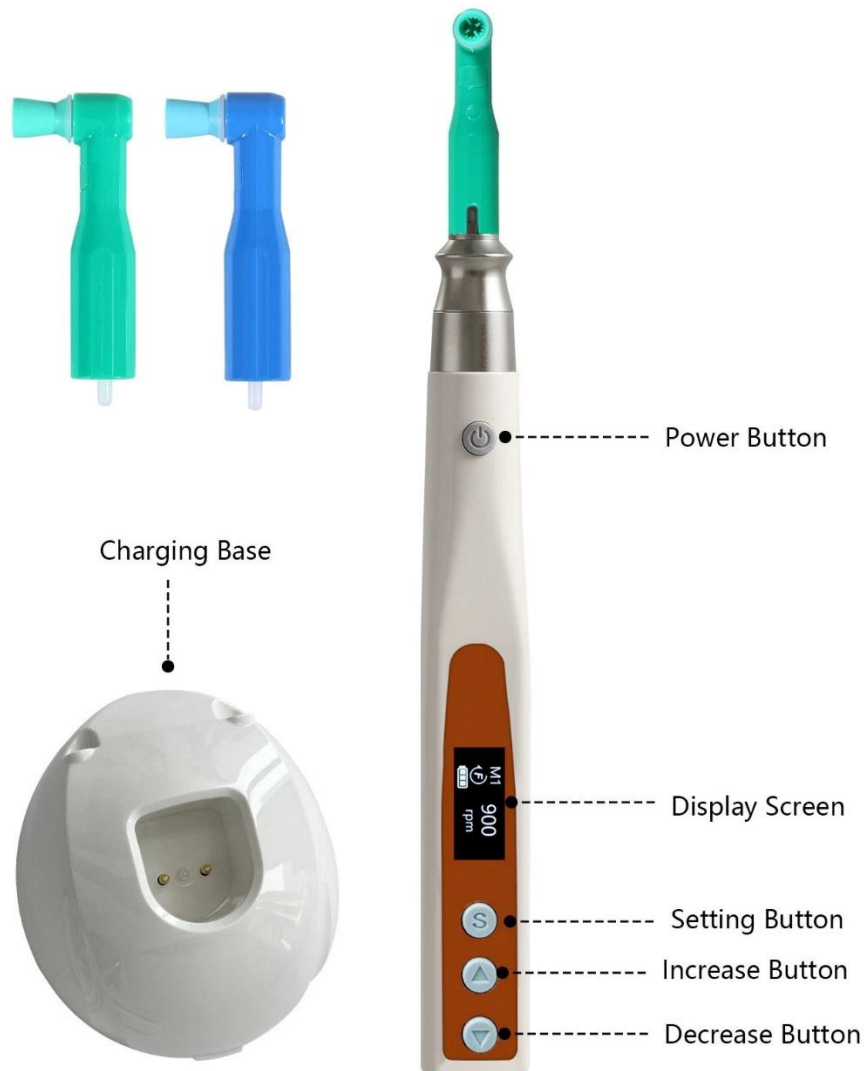

Button Description

- Power Button: Long press the button to turn On or Off the device; When the device is already powered on, a short press of the button can turn the motor On or Off.
- Setting Button: A short press of the button switches between different memory modes; A long press of the button enters the parameter

setting and to save the data of the current memory mode, at this time, a short press of the button is used to select the parameter that needs to be adjusted;

- $\Delta \nabla$ Adjustment Button: Press the $\Delta \nabla$ buttons to adjust the relevant parameters; When not in the setting mode, the $\Delta \nabla$ buttons can only be used to adjust the rotation speed;

Display Description

- There are 9 built-in memory modes from M1 to M9, which can adapt to various different requirements;
- The display of the letter "R" indicates forward rotation, and the display of the letter "F" indicates reverse rotation;
- The rotational speed ranges from 600rpm to 3000rpm;

Cautions:

- Wear appropriate protective glasses during use to prevent debris from flying into the eyes during grinding.
- Ensure the device is used in a dry and clean environment and avoid water or other liquids from entering the device.
- Regularly check the charger and plug of the grinder. Replace them in time if damaged.
- Do not use a damaged charger or use it with a loose socket.

4.Packing List

Motor Handle x 1pc

Straight x 1pc

Charging Base x 1pc

Adaptor+USB Cable x 1set

Operation Manual x 1pc

Prophy Angles x 2pcs

Alandental.com Contact Information

Email: service.alandental@gmail.com

Url: <https://www.alandental.com>